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
Touchless dispenser for soap and disinfectant

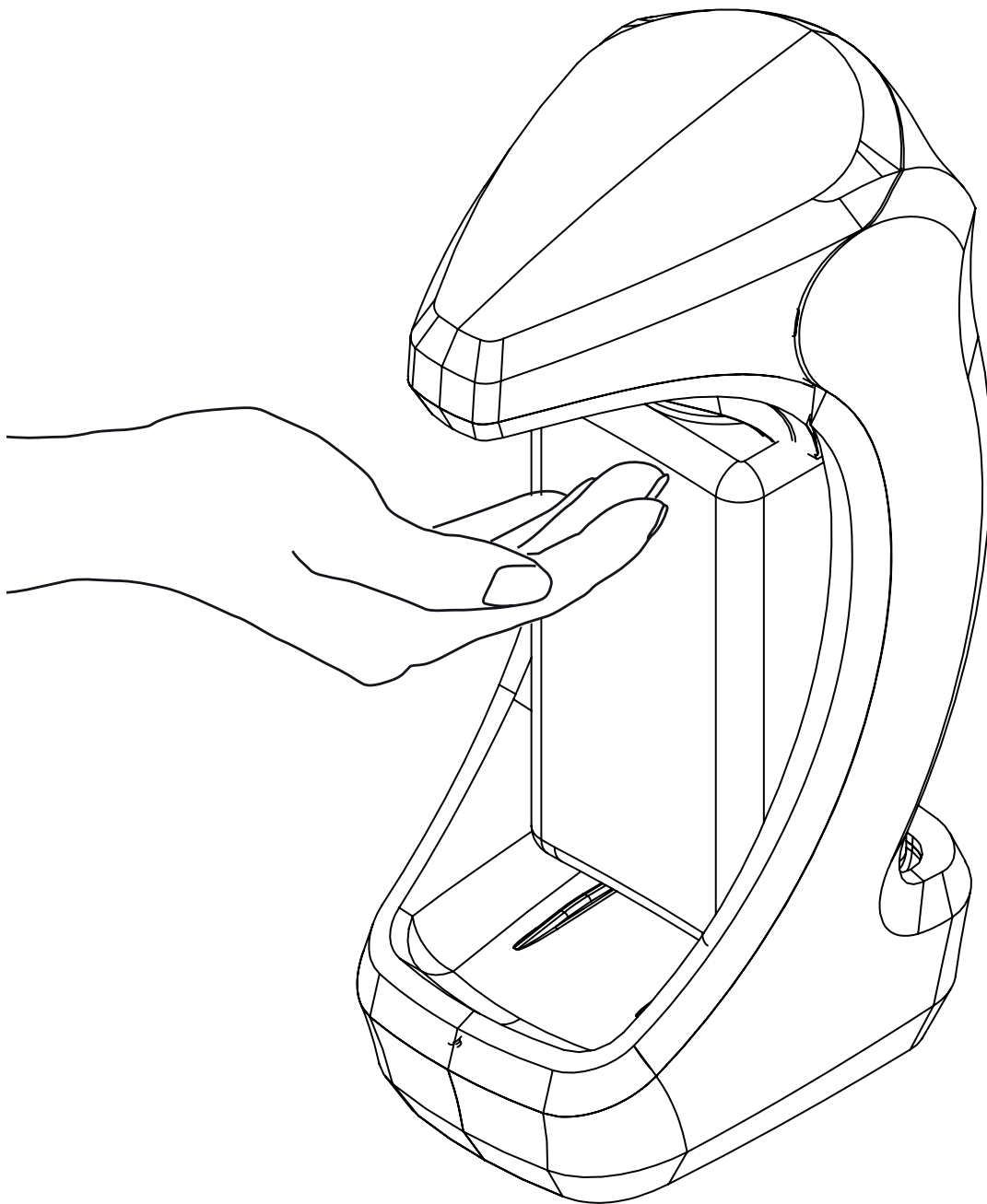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<b>EN - Instructions for use</b> .....	<b>15</b>
<b>1. Notes on the instructions for use</b> .....	<b>15</b>
<b>2. Designated Use</b> .....	<b>15</b>
2.1. Ambient conditions.....	15
<b>3. Safety information</b> .....	<b>16</b>
3.1. Safety.....	16
3.2. Safety during operation of the device .....	17
<b>4. Product description</b> .....	<b>17</b>
4.1. Technical data / performance data.....	17
<b>5. Preparation for use</b> .....	<b>18</b>
5.1. Installation and assembly.....	18
5.1.1. Free-standing installation .....	18
5.1.2. Wall mounting.....	18
5.1.3. Removing the wall bracket .....	20
5.1.4. Attaching the drip tray.....	20
5.2. Commissioning .....	20
5.2.1. Inserting the batteries.....	20
5.2.2. Assembling the pump and bottle .....	21
<b>6. Use</b> .....	<b>23</b>
6.1. Application .....	23
6.2. LED signals.....	23
6.3. Troubleshooting .....	23
<b>7. Upkeep and maintenance</b> .....	<b>24</b>
7.3.1. Cleaning the dispenser.....	24
7.3.2. Cleaning the pump .....	24
<b>8. Disposal</b> .....	<b>25</b>
8.1. Disposal of the batteries .....	25
<b>9. REF numbers and scope of delivery</b> .....	<b>25</b>
<b>10. Guarantee terms</b> .....	<b>25</b>

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# EN - Instructions for use

## IMPORTANT!

Before using new filling goods, the compatibility of the filling goods and the dispenser system must be checked in order to be guarantee functionality. The manufacturer assumes no liability for any damage or costs that may arise due to incompatibilities between filling goods used and the dispenser system.

## 1. Notes on the instructions for use

The following symbols are used in the instructions:

Safety reference:  
Non-observance may cause personal injury

May result in property damage


## 2. Designated Use

This device is suitable for hand disinfectants and liquid soaps. It is not intended for use with high-viscosity products or abrasive and aggressive agents. The dispenser can be used free-standing, or mounted on a wall or a stand. Use this device indoors only. Use the device for hand disinfection and cleaning under normal conditions of use.

### 2.1. Ambient conditions

Never use the device in potentially explosive areas.

Only use the device in a dustless environment.

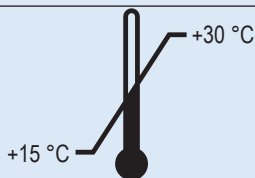Only use this device at room temperature 15 - 30 °C

\*Please also note the manufacturer's information of the product used!

Make sure that the device is only used at a relative humidity of <90% (non-condensing).

## 3. Safety information

### Attention!

Please read these operating instructions carefully before first use and keep them for future reference.

The manufacturer does not assume any liability for damages in the case of:

- > damage to the device caused by mechanical influences
- > modifications of the device without express approval of the manufacturer
- > use for purposes other than those mentioned in the application

### 3.1. Safety

- > Repairs to the device may only be carried out by employees of the manufacturer or by authorized persons of the manufacturer. Improper repairs can result in considerable dangers. In case of repairs, contact the manufacturer.
- > Install the device in a dry and splash-proof place.
- > This device is not a toy. Keep and use it out of the reach of children. The device should be installed where children cannot reach it.
- > Persons with restricted motor, sensory or mental abilities as well as children must be supervised when using the device.
- > The device is not suitable for the transport of filling goods.
- > Do not place the device directly in heavily frequented areas.
- > The device is not suitable for dosing anything other than the specified filling goods.
- > Only use the device with leak-proof alkaline batteries of the same manufacturer and type and with the same charge level. Use only type C batteries.
- > Do not use the device with rechargeable batteries.
- > Do not continue to use the dispenser if the batteries in the device have leaked.
- > Never use brute force.
- > Remove the batteries from the device before cleaning and maintenance.
- > Do not disassemble the device. It does not contain any repairable parts.
- > The outlet cover cannot be removed.
- > The battery terminals must not be short-circuited.

### 3.2. Safety during operation of the device


- > Protect the device from moisture.
- > Do not expose the device to thermal radiation and heat.
- > Do not submerge the device in water.
- > Only fill the device with the specified filling goods in accordance with its intended use.
- > The device has no detachable dismountable parts with the exception of the pump, bottle, battery cover and batteries
- > The device is not autoclavable and cannot be cleaned in the dishwasher.
- > The device must not be cleaned with a water jet or high-pressure cleaner.
- > Make sure that the filling goods do not drip onto the floor.

## 4. Product description

META TOUCHLESS is a dispensing system for contactless, sensor-controlled dosing of hand disinfectants and soaps for 500 ml euro bottles.

### 4.1. Technical data / performance data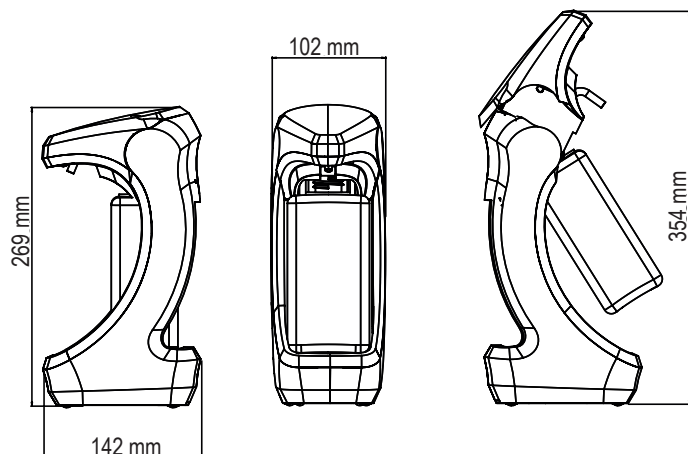

**Technical modifications reserved.**

Test are required when using new filling goods!

Housing	ABS
Bottle	plastic, removable; 500 ml
Pump	PP, PE & silicone
Dosage	1.5 ml / stroke
Voltage	6 VDC
Batteries	Batteries 4 x LR 14
Dimensions (H x B x T)	269 x 102 x 142 mm

Total weight (with batteries, without wall plate)	0.930 kg
Weight with wall plate	1.067 kg
	RoHS compliant
Conformity	 Machinery directive 2006 / 42 / EG EMC- directive 2004/108/EG EN 55014-1:2006 EN 55014-2:2001 EN 60335-1/A11:2014
Manufacturer	Ophardt Ireland, Tubbercurry Road, Ballymote, Co. Sligo, Ireland

## 5. Preparation for use

### 5.1. Installation and assembly

The dispenser can be used free-standing or mounted on a wall.

#### 5.1.1. Free-standing installation

If you are using it free-standing, make sure that:


- > the surface is level and stable
- > there are no moving objects nearby
- > if the dispenser falls over, any leaking product does not come into contact with other objects.

#### 5.1.2. Wall mounting


- > The dispenser must be attached to a straight wall with at least 3 screws using the supplied bracket.
- > Use the supplied or other suitable mounting material for wall mounting.
- > Make sure the wall is suitable before installation!
- > When choosing the location, consider the ease of access for filling!
- > Do not install the device in narrow entrance areas and passages!
- > Microwaves or fluorescent tubes can cause electromagnetic interference. If this occurs, remove the cause of interference. The device should then work properly again.

1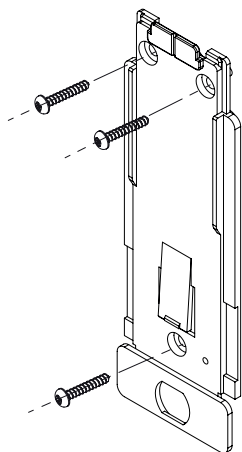


Fasten the wall plate to the wall with the screws and wall plugs provided.

2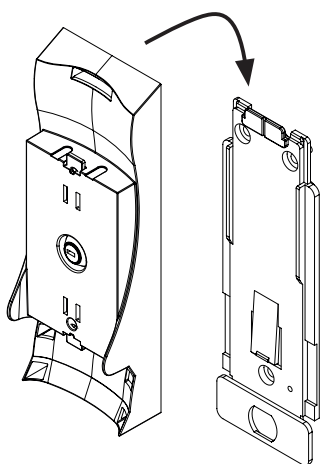


Press the wall bracket against the wall and slide it onto the wall plate until the latch clicks into place.

3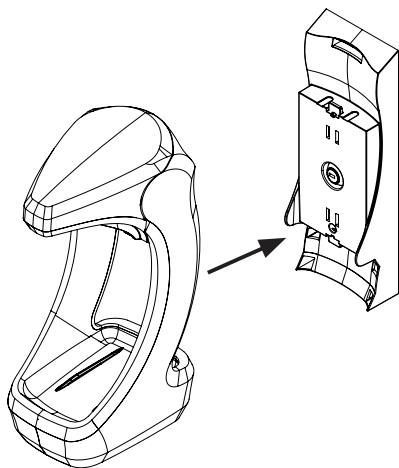

Mount the dispenser on the wall bracket by sliding the opening on the bottle over the rectangular fixture on the wall bracket.


4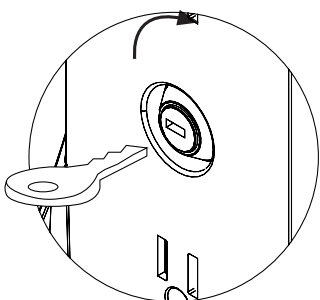

Turn the key to lock the dispenser. Remove the key before inserting the bottle.

### 5.1.3. Removing the wall bracket

---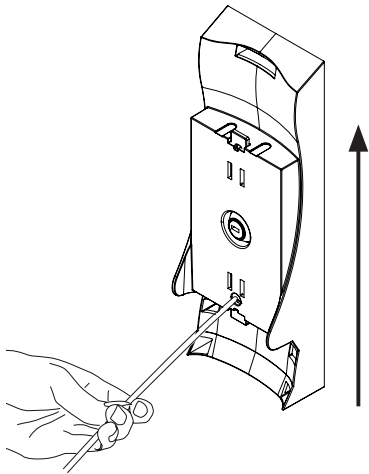

To remove the wall bracket from the wall plate, insert a pointed tool into the hole provided and press the latch of the wall plate backwards. The wall bracket can now be pushed out upwards.

---


### 5.1.4. Attaching the drip tray

If desired, the supplied drip tray can be clipped on.

## 5.2. Commissioning

### 5.2.1. Inserting the batteries


---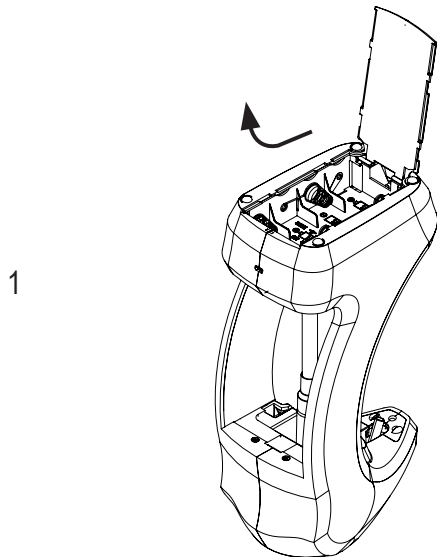

To open the battery compartment, slide the cover towards the back of the dispenser and rotate the cover upwards.

---

2

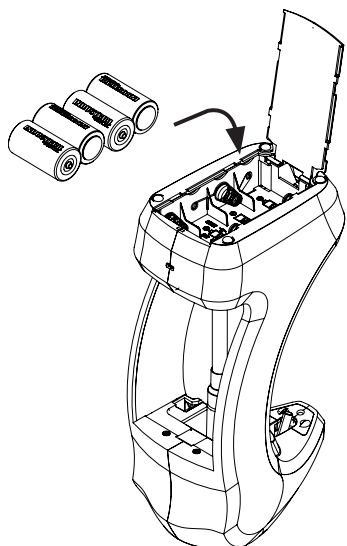Insert the batteries according to the markings in the battery compartment.


Pay attention to correct polarity. Place the tape under the batteries before inserting them.

**Remove the batteries if the device will not be used for longer periods.**


### 5.2.2. Assembling the pump and bottle

1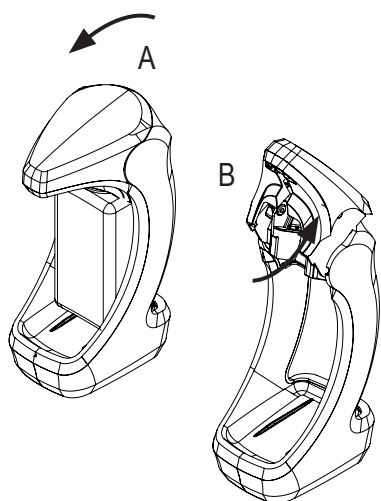


To open the outlet cover, first pull it forwards (A) and then rotate it upwards towards the rear of the dispenser (B).

2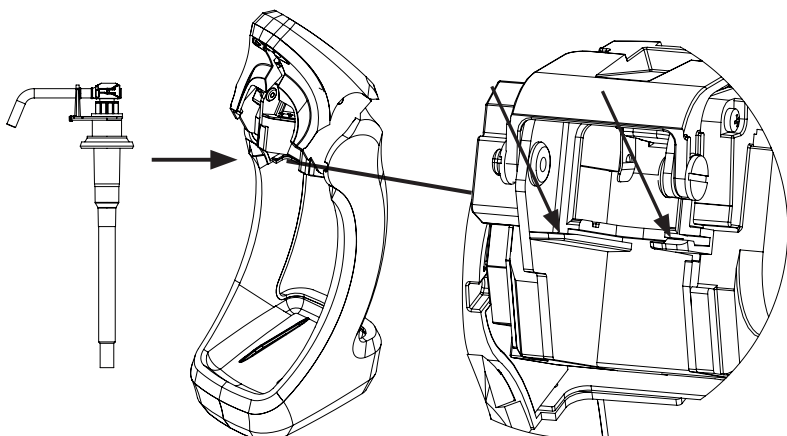


Slide the pump into the guides on the housing and snap the locking tab into place as shown.

3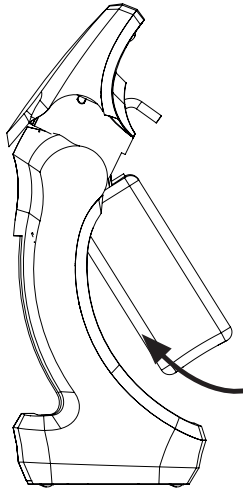

Hold the bottle at a slight angle and slide it over the pump.

4

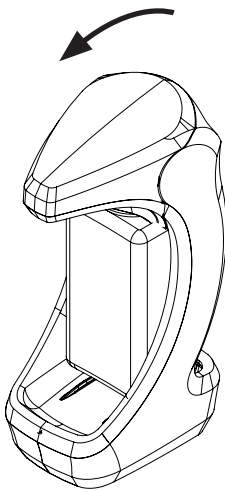
Press down on the outlet cover at the front and make sure that it is closed properly (check the back and sides).

**Remove the contents before transport.**

## 6. Use

### 6.1. Application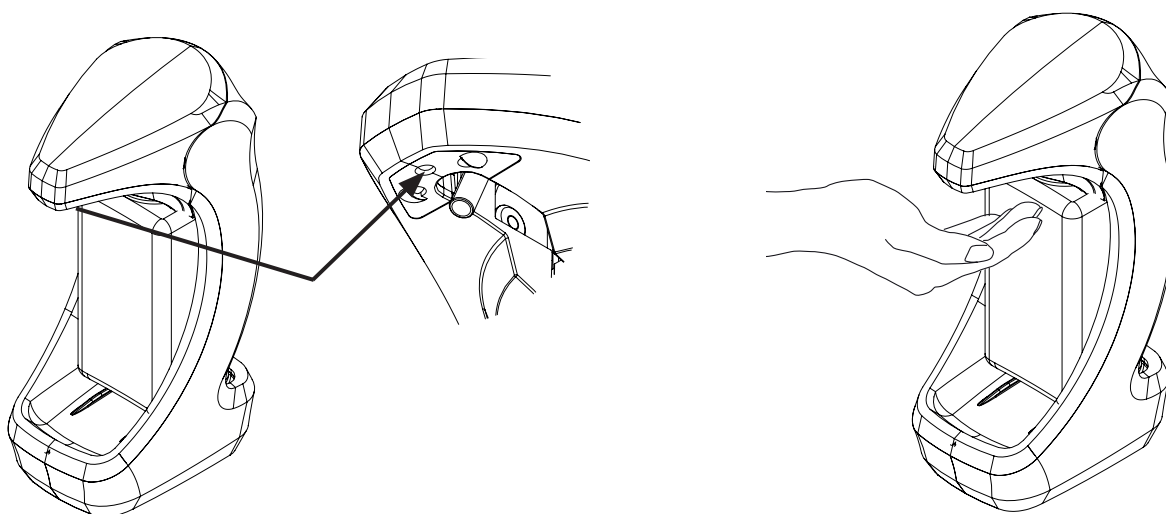


Once the outlet cover is closed, wait 3 seconds for the red LED to go off. To activate the dispenser mechanism, hold your hand under the outlet cover (approx. 5 cm away). This will trigger the dispenser to deliver a stroke.

Dosage: 1.5 ml / spray

### 6.2. LED signals

Below the outlet cover is an LED which lights up green or red.

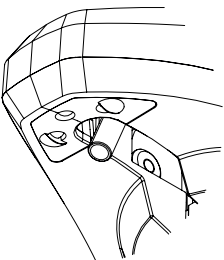	Color	Description
	RED (permanent)	Outlet cover open / not properly closed
	RED (flashing)	Battery low
	GREEN	Delivery of strokes

### 6.3. Troubleshooting

If the device does not work properly, first remove the batteries.

Problem	Malfunction	Possible Cause
The device appears to be working normally, but no liquid is dispensed.	No bottle inserted.	Insert a full bottle.
	No pump used.	Insert the pump into the dispenser.
	Device is not closed.	Close the device.
	The pump is blocked.	Remove and clean the pump.
	The bottle is empty.	Replace the bottle.
	The pump is not primed or has no contact with the liquid.	Perform the dosing process several times. If this does not work, the pump may be clogged.
	An incompatible product is being used.	Use a different product. Thoroughly clean the pump beforehand.
The device does not dispense.	The batteries are empty.	Insert new batteries.
	Batteries have not been inserted correctly.	Pay attention to the correct polarity.

## 7. Upkeep and maintenance

Even a high quality product like this device requires upkeep and maintenance.

The dispenser itself is maintenance-free with the exception of the battery change. Only use hot water for cleaning and do not use any caustic or abrasive cleaning agents.

Follow the recommendations of the Robert Koch Institute ([www.rki.de](http://www.rki.de)).

Please only use hot water and do not use caustic purifiers or scouring powders!

### 7.3.1. Cleaning the dispenser

Remove the cover and take out the batteries. Only clean the outside of the dispenser. Wipe the dispenser with a damp cloth.

Never disassemble the device!

If the bottle is full, a longer period of non-use will not cause any impairments. Therefore, you should replace an empty bottle in a timely manner.

Make sure not to submerge the device or expose it to a water jet.

The device is not autoclavable and cannot be cleaned in the dishwasher.

### 7.3.2. Cleaning the pump

The pump itself is suitable for the dishwasher (up to 60 ° C).

After being cleaned in the dishwasher, it must be dried for about 6 hours before being put back into the device.

If the suction valve is still clogged, pump hot water through it to restore the function.

## 8. Disposal

This symbol on the product or its packaging indicates that this product must not be disposed of with household waste, but must be taken to a recycling point for electrical and electronic equipment. Further information may be obtained from your municipality, the municipal disposal companies or the company from where you purchased the product.

### 8.1. Disposal of the batteries

Batteries must not be disposed of with household waste. As a consumer, you are legally obliged to return used batteries. You can drop off batteries at domestic or public collection points in your city or municipality, or at any location where batteries of this type are sold.

The packaging material can be recycled.

## 9. REF numbers and scope of delivery

Order no.	Description
40200106	META TOUCHLESS Automatic soap and disinfectant dispenser, bottle 500 ml (empty), wall bracket, drip tray, mounting kit (screws and dowels), batteries (4 mono cells, type LR 14), instructions for use

## 10. Guarantee terms

2-year manufacturer's guarantee from the date of delivery.

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**METASYS Medizintechnik GmbH**