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
Dispenser for soap and disinfectant

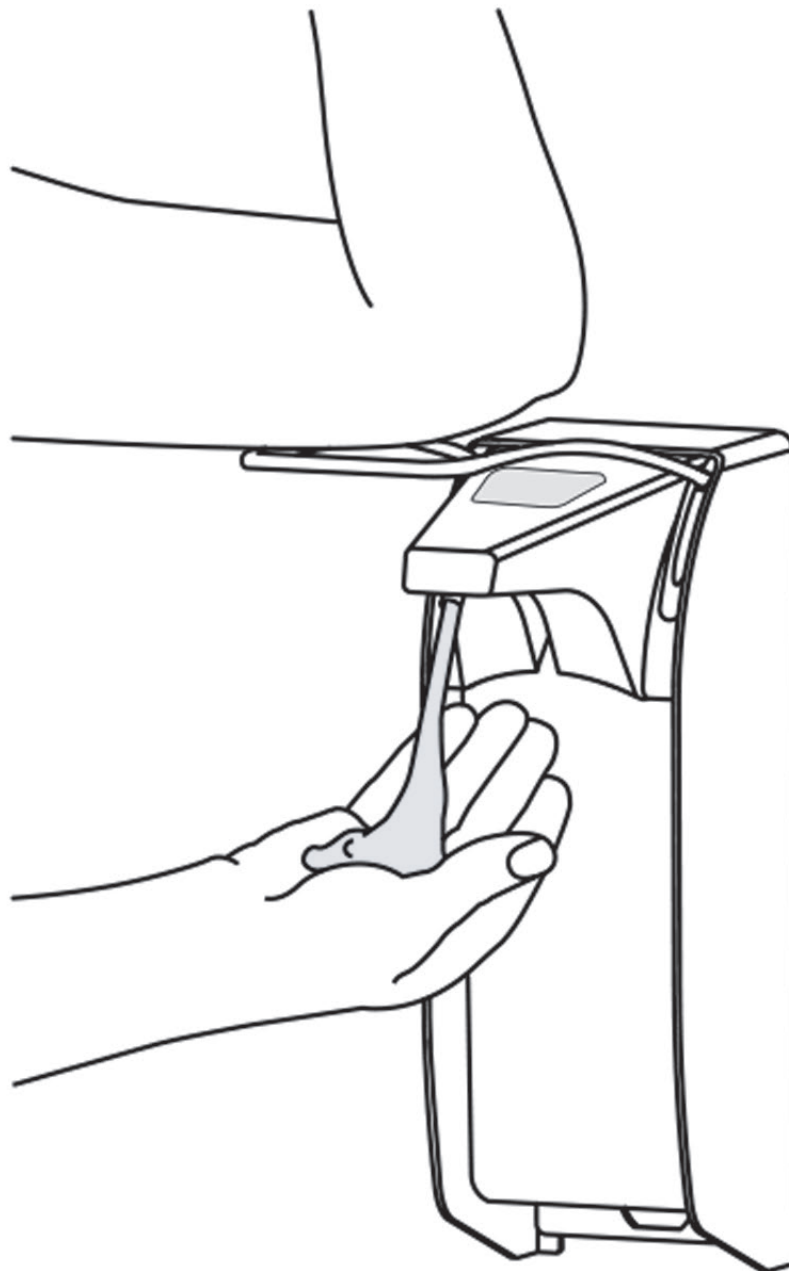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EN - Instructions for use	11
1. Designated Use	11
2. Product description	11
2.1. Technical data / performance data	11
3. Preparation for use	12
4. Assembly	12
4.1. Commissioning	13
4.1.1. Placement / exchange of the bottle	14
4.1.2. Flexible sealing cap	14
5. Use	14
5.1. Application	14
5.2. Pump removal / pump replacement	15
5.3. Setting the dosage	15
6. Upkeep and maintenance	16
6.1. General care instructions	16
6.1.1. Manual cleaning	16
6.1.2. Processing in thermal disinfectant	16
6.1.3. Processing in autoclave (steam sterilizer)	16
7. Disposal	17
8. Annex	17
8.1. REF numbers and scope of delivery	17

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EN - Instructions for use

IMPORTANT!

Before using new filling goods, the compatibility of the filling goods and the dispenser system must be checked in order to be guarantee functionality. The manufacturer assumes no liability for any damage or costs that may arise due to incompatibilities between filling goods used and the dispenser system.

IMPORTANT!


Please read these operating instructions carefully before first use and keep them for future reference.

1. Designated Use

This device is suitable for hand disinfectants and liquid soaps. It is not intended for use with highly viscous, abrasive or aggressive products. Use the dispenser only indoors. The device is intended for hand disinfection and cleaning under normal conditions of use.

2. Product description

2.1. Technical data / performance data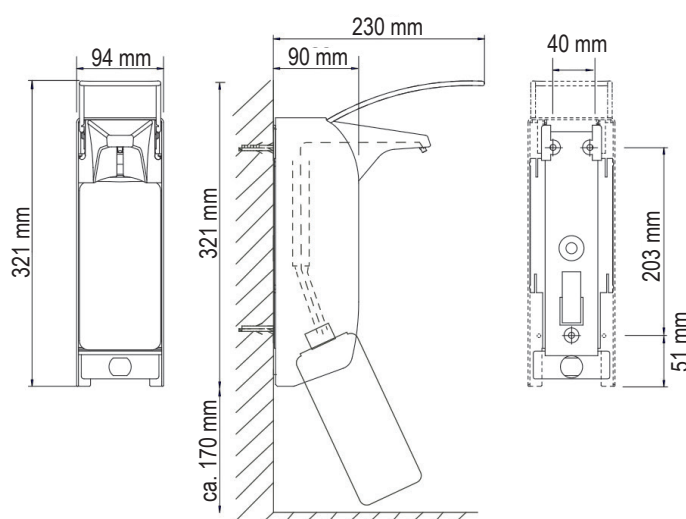

Technical modifications reserved.

Housing	aluminum, matt silver anodized
Bottle	plastic, removable; 1000 ml
Pump	plastic pump
Operating lever	stainless steel
Wall plate	plastic

Wall mounting	wall mounting with 3 screws and dowels
Quantity dispensed	adjustable, 1.5 ml / stroke (depending on the viscosity of the product)
Cleaning	dispenser dishwasher safe and autoclavable up to 134 ° C, 2 bar
Manufacturer	Ophardt Ireland, Tubbercurry Road, Ballymote, Co. Sligo, Ireland

3. Preparation for use


- > When using as a soap dispenser, install the dispenser as close as possible to the sink to prevent wet hands from dripping onto the floor.
- > Required space above the dispenser: 50-70 mm for the lever, and for dispenser removal and placement, as well as an additional 100 mm for operation. The required room below the dispenser, necessary for bottle removal, is about 170 mm.
- > Optimal mounting height for use: 130-135 cm, measured from the floor to the upper edge of the dispenser housing.

4. Assembly


1


First remove the bottle: Tilt the bottom of the inserted bottle slightly forwards and remove the bottle from the dispenser.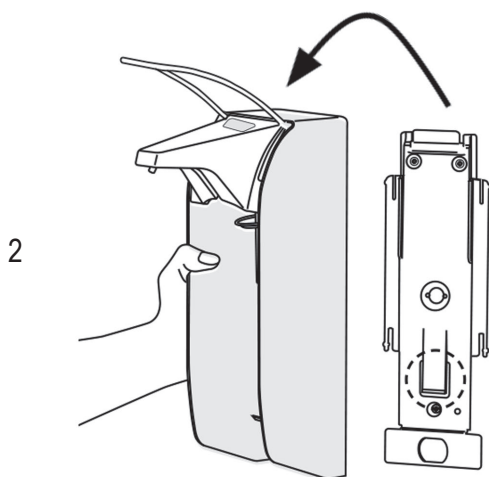

Remove the dispenser from the wall plate: Press the latch on the wall plate and slide the dispenser up and out.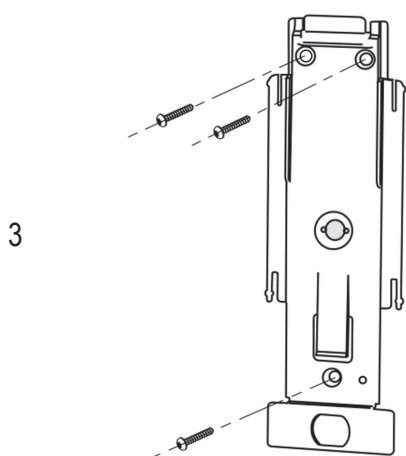

Fasten the wall plate to the wall with the enclosed or other suitable fastening material.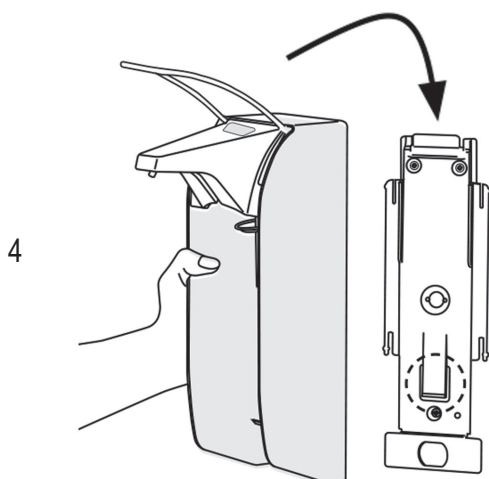


Slide the dispenser onto the wall plate from the top until the latch clicks into place.

4.1. Commissioning

When using for the first time and after replacing the pump or the bottle, press the lever a few times as far as it will go until the dispenser doses. Hold your hand under the spout.

4.1.1. Placement / exchange of the bottle

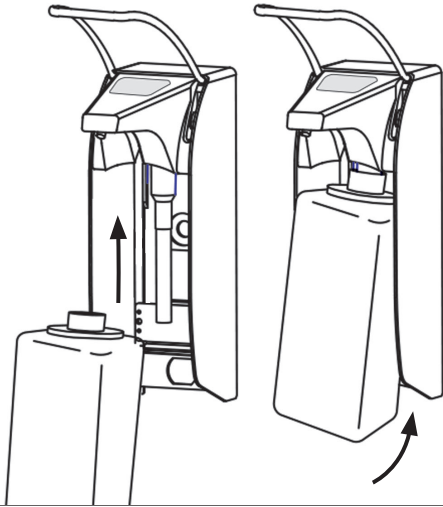Insert the filled bottle into the dispenser from below until it clicks into place.

4.1.2. Flexible sealing cap

The flexible sealing cap included in the scope of delivery ensures optimum pressure for different bottle sizes and prevents the loss of alcohol from the disinfectant.


Use only for liquid soap and disinfectant pumps.

5. Use


5.1. Application

The product is dosed by pressing the lever. The dispenser is preset to a maximum dosage of approx. 1.5 ml / stroke. To change the dosage, see „5.3. Setting the dosage“.

5.2. Pump removal / pump replacement

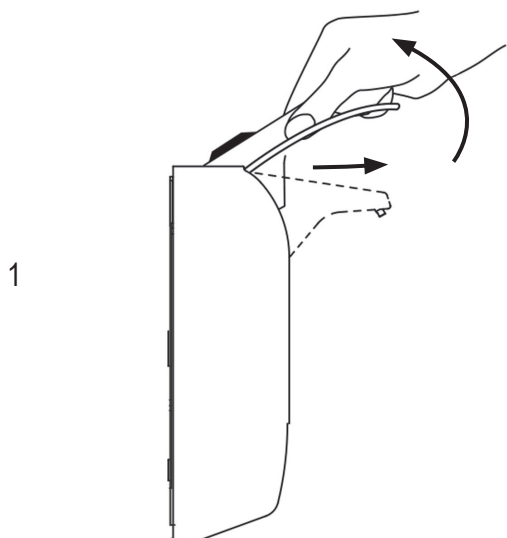
First pull the outlet cover slightly forward and then rotate it upwards.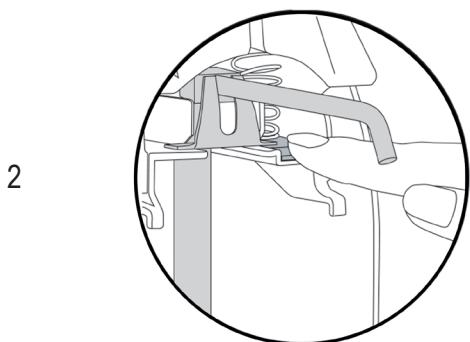

Hold down the pump lock with your finger and pull the pump out towards you.

5.3. Setting the dosage

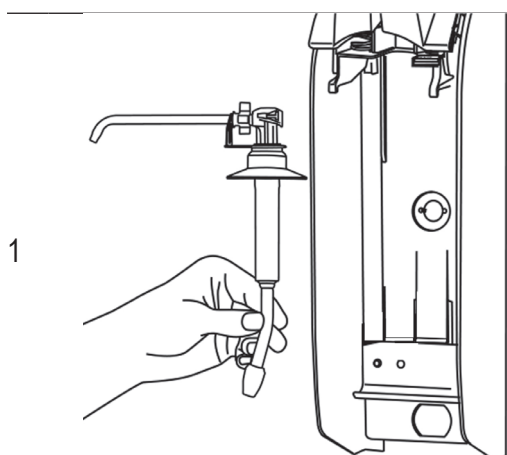To change the dosage, first remove the pump from the dispenser (see „6.2. Pump removal / pump replacement“).

2

The dosage can be changed to 1.5 ml (MAX) / 1.0 ml / 0.75 ml (MIN) (dosage dependent on the viscosity of the filling solution) by turning the adjustable wheel.

With first use, the wheel may be hard to turn; if necessary use a pair of flat-nose pliers.

IMPORTANT:

Do not dismantle the housing! Use only original spare parts. The dispenser is only suitable for dispensing compatible products such as liquid soap (without abrasives), low viscosity lotions or disinfectants.

The use of incompatible products can impair the function of the dispenser and may cause liquid to spray into eyes!

6. Upkeep and maintenance

6.1. General care instructions

The Robert Koch-Institute recommends: "For both dispenser types (note: hand disinfectant and soap dispenser), the fixed outer and inner parts must be able to be reprocessed. This is also possible by machine. (...) Visible contamination of the outer housing and all accessible parts without further manipulation as well as soilings at the outlet must be removed by wipe disinfection. Daily wipe disinfection is recommended for operating levers. The scope and frequency of checks and the reprocessing of dispensers should be specified in the internal hygiene plan, e.g. every six months." (Source: Robert Koch Institute, recommendations for hand hygiene, 2016)

In order to guarantee proper functioning of the dispenser regular manual cleaning is necessary.

6.1.1. Manual cleaning

Regularly remove adhering product residues. Clean the dispenser housing and the pump regularly with hot water. Rinse the pump with hot water. Disinfection of the housing is also possible with suitable surface disinfectants.

6.1.2. Processing in thermal disinfectant

Choose program „Dispenser cleaning” and use pH-neutral cleaning solutions. Important: Cleaning of plastic pumps up to max. 65 °C!

6.1.3. Processing in autoclave (steam sterilizer)

The dispenser is autoclavable with superheated steam (134 °C, 2 bar, max. 20 min dwell time).

**IMPORTANT:**

Wall plate, bottle and plastic pumps (K pumps) are not autoclavable!

After autoclaving, let the dispenser cool for at least one hour prior to next use!

7. Disposal

The packaging material can be recycled.

The device can be disposed of at a local collection point. Further information may be obtained from your local authority, waste disposal service or the company from where you purchased this product.

8. Annex

8.1. REF numbers and scope of delivery

Order no.	Description
40200105	META TOUCH Soap and disinfectant dispenser, bottle 1000 ml (empty), wall plate, mounting kit (screws and dowels), instructions for use

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METASYS Medizintechnik GmbH