

BR Biofilm-Reinigungsgerät

Gerätedokument
Einbau, Betrieb und Wartung

BR Biofilm removing system

Equipment Logbook
Assembly, operation and maintenance

BR Système d'élimination du biofilm

Livret d'appareil
Installation, fonctionnement et entretien

BR Sistema eliminazione biofilm

Verbale d'installazione
Montaggio, funzionamento e manutenzione

BIOFILM REINIGUNGSGERÄT



METASYS

EN

BR Biofilm removing system · GREEN&CLEAN BR · Scope of delivery

Practice personnel, technicians



1

1. BR Biofilm removing system

The METASYS BR biofilm removing device eliminates biofilm from the tubing system of the dental unit.

Product features

- Simple connection
- Easy to handle (pumps the cleaning solution automatically into the particular dental unit)
- Action time may range between half an hour up to max. 1 hour
- Only to be used in combination with METASYS GREEN&CLEAN BR
- Depending on the dental unit the usage of GREEN&CLEAN BR ranges from 500 ml up to 1000 ml (approx. equivalent to one bottle)
- Biologically degradable according to the EWG-guideline 84/449
- 230 volt operating voltage
- CE mark



2

2. GREEN&CLEAN BR

4% solution on hydrogen peroxide basis for the removal of biofilm in dental units. In combination with the METASYS BR biofilm removing system, GREEN&CLEAN BR is recommended for the refurbishment of dental units as well as before the installation of a water decontamination system.

Spectrum of efficacy: For the removal of biofilm in dental units.

Safety recommendation: Avoid contact with skin and eyes. Keep away from children. Do not swallow!

The preparation is not dangerous in the sense of Directive 1999/45/EC. This mixture is classified as not hazardous according to Regulation (EC) 1272/2008 [GHS].

Composition: 4% hydrogen peroxide, water, colourant.



3

3. Scope of delivery

BR Biofilm cleaning and removing kit
Order no.: 40050501

Biofilm removing system + 1 x 1000 ml bottle of GREEN&CLEAN BR cleaning solution

GREEN&CLEAN BR
Order no.: 60040300


2 x 1000 ml bottles of GREEN&CLEAN BR cleaning solution

Application

Practice personnel, technicians

4. Application

- 4.1** Remove all water consumers from the dental unit (to prevent clogging).
Remove the black cap.
- 4.2** Connect the supplied GREEN&CLEAN BR bottle with the connection adapter.
- 4.3** Disconnect the sealing plug.
- 4.4** Connect the water inlet of the dental unit to the METASYS BR biofilm removing device.
- 4.5** Open the inlet tap.
- 4.6** Switch on the device.
- 4.7** As soon as the violet liquid exits from the water consumers, switch off the device again.
- 4.8** Close the inlet tap.
- 4.9** Disconnect the water inlet of the dental unit from the device. Remove the connection adapter from the GREEN&CLEAN BR bottle, and put the black cap and the sealing plug back on.
- 4.10** The exposure time lasts between 30 to max. 60 minutes, depending on the contamination of the dental unit. After completion of the exposure time, actuate all water consumers until clear water emerges.

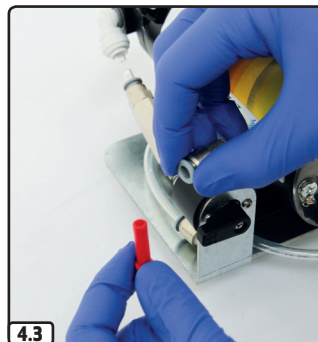

METASYS ... makes the difference!

Visit us at:



METASYS Medizintechnik GmbH
Florianstraße 3, 6063 Rum bei Innsbruck, Austria

☎ +43 512 205420 | 📞 +43 512 205420 7

www.metasys.com | info@metasys.com



GERMANY
+49 8823 938 44 33
info@metasys.com

FRANCE
+33 4 37 90 22 15
info@metasys.fr

ITALY
+39 045 981 4477
desitalia@metasys.com