GREEN&CLEAN CLN

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

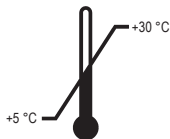
1.1. General instructions

Storage temperature

Product identifiers

Alkylethercarbonic acid (53563-70-5)
 Protease (9014-01-1)
 Lipase (9001-62-1)

1.2. Explanation of the symbols

| | | | |
|----|---------------------------------|----|--|
| 1 | CE mark | 14 | Observe instructions for use |
| 2 | Medical device | 15 | Use hand protection |
| 3 | Item number | 16 | Use eye protection |
| 4 | Batch designation | 17 | Wear protective clothing |
| 5 | Expiry date | 18 | Information |
| 6 | Manufacturer | 19 | Download |
| 7 | Follow instructions for use | 20 | Protect from heat / Protect from sunlight |
| 8 | Temperature limit | 21 | Protect from moisture / Store in a dry place |
| 9 | Quantity information | 22 | Fragile |
| 10 | Responsible for the translation | 23 | Package orientation at the top |
| 11 | Health hazard | 24 | Limited quantity |
| 12 | Corrosive | 25 | Stacking limit |
| 13 | Harmful to health | 26 | Name and address of the authorized representative's registered office in Switzerland |

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended use

GREEN&CLEAN CL N is a concentrate for cleaning and deodorising the suction lines of dental suction systems.

Indication: Removal of blood and saliva in the waste water path of the dental suction system. To be used at the end of the working day and after long surgical procedures.

Contraindication: Not applicable.

Target group: Patients and trained medical personnel, without restriction as to gender and age, who come into contact with suction lines of dental suction systems.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

Warning

| | |
|--------------------|--|
| H318 | Causes serious eye damage. |
| H334 | May cause allergy or asthma symptoms or breathing difficulties if inhaled. |
| H315 | Causes skin irritation. |
| P101 | If medical advice is needed, have product container or label at hand. |
| P102 | Keep out of reach of children. |
| P261 | Avoid breathing dust/fume/gas/mist/vapours/spray. |
| P280-metasy | Wear protective gloves/protective clothing/eye protection/face protection. |
| P304 + P340 | IF INHALED: Remove person to fresh air and keep comfortable for breathing. |
| P342 + P311 | If experiencing respiratory symptoms: Call a POISON CENTER/doctor. |
| P305 + P351 + P338 | IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. |
| P310 | Immediately call a POISON CENTER/doctor/... |
| P501 | Dispose of contents/container to a suitable recycling or disposal facility. |

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

| | |
|-----------------------------|--|
| General instructions | Immediately remove contaminated, saturated clothing. |
| After inhalation | In the event of breathing difficulties, give oxygen. In the event of irregular breathing or respiratory arrest, initiate artificial respiration. Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label. |
| After skin contact | In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water. |
| After eye contact | In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist. |
| After ingestion | Rinse mouth immediately and drink plenty of water. Do not induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately. |

3.4. Personal protective equipment (PPE)

| | |
|----------------------------|--|
| Eye/Face protection | Suitable eye protection: Tightly sealing safety goggles Recommended eye protection brands: DIN EN 165 |
|----------------------------|--|

| | |
|-------------------------------|--|
| Hand protection | Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm Recommended glove brands: EN ISO 374 Additional hand protection measures: Wear cotton undershirts if possible. |
| Physical protection | Suitable physical protection: Laboratory coat |
| Respiratory protection | Respiratory protection is not required when used properly and under normal conditions. |

4. Product description

4.1. Product description

GREEN&CLEAN CL N is a concentrate based on alkylethercarboxylic acid, protease and lipase for daily cleaning of suction systems.

4.2. Composition

Alkylethercarboxylic acid combined with protease and lipase

4.3. Exposure time

The user solution is sucked in by the suction flow of the suction unit and should remain in the suction system for 60 minutes if possible. Longer exposure times are also possible (e.g. over the weekend).

5. Use

5.1. Dosage

| Water | Solution |
|--------|----------|
| 0,60 l | 1.6 % |

5.2. Application

Note

- > Rinse the suction system with cold water after each treatment
- > Wear personal protective equipment during application
- > Adhere to the dosage
- > Mixed solution is intended for immediate use

Application

Open the sealing cap of the dispensing container on the top of the bottle.

27 Fill the container with 10 ml of the concentrate by gently pressing the bottle.

Only dispense when the sealing cap is open.

Empty the 10 ml GREEN&CLEAN CL N into the mixing container.

28 Fill the container up to the mark with cold water (= 600 ml 1.6% solution)

29 Hold the small suction hose on opening no. 1 until air is sucked in (200 ml)

30 Hold the large suction hose on opening no. 2 until air is sucked in (200 ml)

31 Empty the remaining solution into the rinsing basin (200 ml). Do not rinse.

32 Wait for action time (60 min) and then rinse with cold water

Allow GREEN&CLEAN CL N to work overnight and over the weekend. If necessary (e.g. in case of high soiling), we recommend an additional application at noon.

6. Disposal

Dispose of product in compliance with official regulations.

| | before intended use | after intended use |
|--------------------|-------------------------------|--------------------|
| | Product | Packaging |
| Product waste code | 70699 | 150102 |
| Hazardous waste | No | No |
| Waste designation | Waste not otherwise specified | Plastic packaging |

7. Annex

7.1. Order numbers and scope of delivery

| Order number | Designation |
|--------------|---------------------------------------|
| 60010305 | GREEN&CLEAN CL N, 2 x 1000 ml bottles |
| 40200023 | Mixing container for GREEN&CLEAN CL N |

7.2. Change history

| Revision | Date | Description |
|--------------|------------|---|
| ZK-58.102/01 | 28.06.2022 | Correction: Chapter 1.2 „REF number“ changed to „item number“ |
| ZK-58.102/00 | 18.06.2021 | Newly created. |

Corrected various spelling and translation errors.

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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