GREEN&CLEAN IK

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.


1. Notes

1.1. General instructions

UN 1950
DENTAL USE

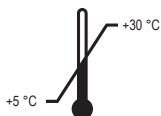Storage temperature

Product identifiers

1-propanol [71-23-8]
2-propanol [67-63-0]
2-butanol [78-92-2]

1.2. Explanation of the symbols

1	CE mark with notified body	13	Use eye protection
2	Medical device	14	General warning sign
3	Item number	15	Information
4	Batch designation	16	Protect from heat / Protect from sunlight
5	Expiry date	17	Protect from moisture / Store in a dry place
6	Manufacturer	18	Fragile
7	Caution / attention	19	Package orientation at the top
8	Temperature limit	20	Limited quantity
9	Quantity information	21	Health hazard
10	Responsible for the translation	22	Highly inflammable / highly flammable
11	Observe instructions for use	23	Corrosive
12	Use hand protection		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended use

GREEN&CLEAN IK is a ready-to-use, alcohol-based disinfectant spray for the spray water and spray air channels of dental hand and angle pieces as well as turbines.

Indication: Reprocessing by rinsing with alcoholic disinfectant using adapters provided and permanently mounted for this purpose.

Contraindication: Not applicable.

Intended users: Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

Danger

H224	Extremely flammable liquid and vapour.
H318	Causes serious eye damage.
H336	May cause drowsiness or dizziness.
P201	Obtain special instructions before use.

P202	Do not handle until all safety precautions have been read and understood.
P210	Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P233	Keep container tightly closed.
P271	Use only outdoors or in a well-ventilated area.
P280	Wear protective gloves/protective clothing/eye protection/face protection.
P305 + P351 + P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor/...
P403 + P235	Store in a well-ventilated place. Keep cool.
P405	Store locked up.
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. Risks and side effects

This product has no direct skin contact with the user or the patient.

Danger and safety information for the user, see point 3.2.

3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

After inhalation	After inhalation of spray, seek medical advice immediately and show the packaging or label.
After skin contact	After contact with skin, wash immediately with plenty of soap and water
After eye contact	If the product gets into the eyes, rinse immediately with plenty of water for at least 5 minutes with eyes open. Then consult an ophthalmologist. Protect uninjured eye.
After ingestion	Do not induce vomiting. Rinse mouth immediately and drink plenty of water. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.
Self-protection of the first aider	First aider: Pay attention to self-protection!

3.5. Personal protective equipment (PPE)

Eye/Face protection	Suitable eye protection: Safety glasses.
Hand protection	Suitable glove type: Disposable gloves.

4. Product description

4.1. Product description

GREEN&CLEAN IK is a ready-to-use disinfectant for decontaminating the spray water and spray air channels of hand and angle pieces as well as turbines.

4.2. Composition

Alcohol-based disinfecting solution.

4.3. Spectrum of action and exposure time

Spectrum of action	Standard	Test information	Exposure time
bactericidal	EN 13727	high load	1 min
levurocidal	EN 13624	high load	1 min
tuberculocidal	EN 14348	high load	1 min
germ carrier test bactericidal	EN 14561	high load	2 min
germ carrier test levurocidal	EN 14562	high load	1 min
germ carrier test tuberculocidal	EN 14563	high load	1 min
limited virucidal	EN 14476 EN 16777	high load	30 sec

5. Use

5.1. Application

Rinse the transfer instrument with water after use on the patient.

Place the hand piece, angle piece or turbine in a paper towel to avoid aerosols.

24+25 Place the hand piece, angle piece or turbine on the adapter and flood the spray water and spray air channels completely with GREEN&CLEAN IK.

The spray can is under pressure!

Do not operate the spray head until the transfer instrument is properly attached.

Apply over a wash basin.

6. Care and maintenance

6.1. Regular cleaning measures

The application of GREEN&CLEAN IK does not replace the necessary care process with oil, but can only precede it.

Lubrication of the O-rings

26+27 To lubricate the O-rings, place the GREEN&CLEAN IK adapter in the palm of your hand (wear gloves), spray briefly and wet the palm with the liquid.

7. Disposal

Dispose of product in compliance with official regulations.

	before intended use	after intended use
	Product	Packaging
Product waste code	70699	70699
Hazardous waste	No	No
Waste designation	Waste not otherwise specified	Waste not otherwise specified

8. Annex

8.1. Order numbers and scope of delivery

Order number	Designation
60030182	GREEN&CLEAN IK for coupling hand and angle piece couplings (universal), 2 x 200 ml spray cans incl. pre-mounted adapter
60030183	GREEN&CLEAN IK for turbine coupling W&H, 2 x 200 ml spray cans incl. pre-mounted adapter
60030184	GREEN&CLEAN IK for turbine coupling Bien Air, 2 x 200 ml spray cans incl. pre-mounted adapter
60030185	GREEN&CLEAN IK for turbine coupling KaVo, 2 x 200 ml spray cans incl. pre-mounted adapter
60030186	GREEN&CLEAN IK for turbine coupling Dentsply Sirona, 2 x 200 ml spray cans incl. pre-mounted adapter
60030187	GREEN&CLEAN IK for turbine coupling Castellini, 2 x 200 ml spray cans incl. pre-mounted adapter
60030188	GREEN&CLEAN IK for turbine coupling NSK, 2 x 200 ml spray cans incl. pre-mounted adapter
60030189	GREEN&CLEAN IK for hand and angle piece couplings Dentsply Sirona T1 Classic, 2 x 200 ml spray cans incl. pre-mounted adapter
60030190	GREEN&CLEAN IK for turbine coupling NSK Quick, 2 x 200 ml spray cans incl. pre-mounted adapter

8.2. Change history

Revision	Date	Description
ZK-58.110/00	19.05.2022	Newly created.

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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