
DENTAL USE

**MD**

CE 0044  
+30 °C

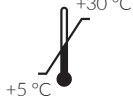# GREEN&CLEAN AD

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## EN – Instructions for use

### Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

**Images:** The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

#### 1. Notes

##### 1.1. General instructions

Storage temperature: +5 - 30 °C

##### 1.2. Product identifiers

2-aminoethanol [141-43-5], N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine [2372-82-9], N,N-Didecyl-N-methyl-poly(oxethyl)ammonium propionate [94667-33-1]

##### 1.3. Explanation of the symbols

1 - CE mark with notified body; 2 - Medical device, 3 - Item number; 4 - Batch designation; 5 - Expiry date; 6 - Manufacturer; 7 - Temperature limit; 8 - Quantity information; 9 - Name and address of the authorized representative's registered office in Switzerland; 10 - Observe instructions for use; 11 - Use hand protection; 12 - Use eye protection; 13 - Protect from heat / Protect from sunlight; 14 - Protect from moisture / Store in a dry place; 15 - Fragi-

le; 16 - Package orientation at the top; 17 - Limited quantity; 18 - Stacking limit; 19 - Health hazard; 20 - Dangerous for the environment; 21 - Unique identifier of a medical device; 22 - UDI marking with standard compliant HIBC data content.

#### 1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

#### 2. Intended use

GREEN&CLEAN AD is a ready-to-use solution for immersion disinfection of dental prosthetic work, worn dentures and dental impressions.

**Indication:** Disinfection of dental prosthetics, worn dentures and dental impressions. **Contraindication:** Observe information on material compatibility in IFU (instructions for use).

**User target group:** Professionally trained health workers.

#### 3. Safety-related information

##### 3.1. General safety-related information

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

#### 3.2. Danger and safety information

**WARNING:** The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP). Causes skin irritation. Causes serious eye irritation. Toxic to aquatic life with long lasting effects. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Do not get in eyes, on skin, or on clothing. Wash thoroughly after handling. Do not eat, drink or smoke when using this product. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. Dispose of contents/container to a suitable recycling or disposal facility.

##### 3.3. Risks and side effects

This product has no direct skin contact with the user or the patient. Danger and safety information for the user, see point 3.2.

##### 3.4. First aid measures

**Emergency number:** Munich Poison Control +49 (0)89 19240

**General instructions:** Immediately remove contaminated, saturated clothing. Seek medical advice if symptoms occur or in case of doubt.

**After inhalation:** In the event of irritation of the respiratory tract, consult a doctor.

**After skin contact:** In case of skin irritation,

consult a doctor. After contact with skin, wash immediately with plenty of soap and water.

**After eye contact:** Rinse immediately carefully and thoroughly with eye-bath or water. If eye irritation occurs, consult an ophthalmologist.

**After ingestion:** Rinse mouth immediately and drink plenty of water.

### 3.5. Personal protective equipment (PPE)

#### Eye/Face protection:

Suitable eye protection: Frame glasses with side shields.

Recommended brands of eye protection: DIN EN 166.

#### Hand protection:

Suitable gloves type Single-use gloves.

Suitable material: NBR (nitrile rubber).

Required properties: liquid-tight.

Thickness of the glove material: 0.4 mm.

Additional hand protection measures: Wear cotton undergloves if possible.

### 3.6. Combination with other products / materials

Immersion tests with polyether, polysulphide and silicone showed very good material compatibility with GREEN&CLEAN AD. The application of GREEN&CLEAN AD to alginates and gypsum may lead to a slight discolouration of the material as well as to a slight deviation in mass (alginate < 1%, gypsum <2.5% at the recommended exposure time of max. 5 minutes).

### 4. Product description

GREEN&CLEAN AD is a highly effective,

ready-to-use solution for quick and lasting immersion disinfection of all prosthetic work and worn dentures, as well as dental impressions.

### 4.1. Composition

Disinfectant solution made from quaternary ammonium compounds, alkylamine, non-ionic surfactants, corrosion inhibitor and water.

### 5. Use

GREEN&CLEAN AD is a ready-to-use solution and is used undiluted.

#### 5.1. Application

**A**- Pour GREEN&CLEAN AD into a suitable (not too large) tub.

**B**- Thoroughly rinse off all visible dirt on the impression with water.

**C**- Rinse the impression with water immedia-

Spectrum of action and exposure time	Test germ	Standard	Test information	Exposure time
bactericidal	<i>S. aureus</i> , <i>E. hirae</i> , <i>P. aeruginosa</i>	EN 13727	high load <sup>1</sup>	2 min
levurocidal	<i>C. albicans</i>	EN 13624	high load <sup>1</sup>	2 min
germ carrier test bactericidal	<i>S. aureus</i> , <i>E. hirae</i> , <i>P. aeruginosa</i>	EN 14561	high load <sup>1</sup>	5 min
germ carrier test levurocidal	<i>C. albicans</i>	EN 14562	high load <sup>1</sup>	5 min
limited virucidal	<i>Vacciniavirus</i>	EN 14476	low load <sup>2</sup>	15 sec
		EN 16777	low load <sup>2</sup>	1 min

<sup>1</sup> Bovine albumin 3g/l + sheep erythrocytes 3ml/l

<sup>2</sup> Bovine albumin 0.3g/l

tely after removing it from the patient's mouth and immerse it completely and bubble-free in the disinfectant solution.

**D** - Wait for the reaction time (5 minutes for all materials).

**E** - Then rinse the impression thoroughly with water.

**F** - Service life: 7 days (replace solution immediately if heavily soiled). In the case of heavy organic pollution, it is recommended to renew the solution more frequently.

**If the denture or prosthesis is reinserted in the patient's mouth, a fresh disinfection solution must be used!**

#### **Instructions for use:**

- Rinse the patient's mouth hygienically before taking the impression.
- Observe the information provided by both manufacturers (impression material and disinfectant) on exposure time and material compatibility.
- Use a suitable tub for impression disinfection: Instrument disinfection tubs often have a large capacity and therefore consume large amounts of disinfectant.
- The exposure time should not exceed 5 minutes (shape stability)!
- Make a written agreement with the laboratory to avoid double disinfection.

- Wear safety gloves!

#### **6. Disposal**

Dispose of product in compliance with official regulations.

##### **• before intended use:**

Waste code Product: 070699; Hazardous waste: No; Waste designation: Waste not otherwise specified

##### **• after intended use:**

Waste code Packaging: 70699; Hazardous waste: No; Waste designation: Waste not otherwise specified

#### **7. Annex**

##### **7.1. Order numbers and scope of delivery**

**122000001** - 1 x GREEN&CLEAN AD, 5000 ml

##### **7.2. Application aids**

**121000003** - 1 x Disinfection tray with cover and drip-off insert

**121000001** - 1 x Outlet tap for 5 L canister

##### **7.3. Change history**

Revision 200005326v01, 25.08.2023: Changes in contraindication, material compatibility, efficacy table.

Revision ZK-58.114/00, 09.11.2022: Newly created.

Corrected various spelling and translation errors. 28.06.2023

Subject to printing and setting errors!

### Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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