# GREEN&CLEAN SK

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## EN – Instructions for use

### Translation of the original operation instruction

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

#### Images:

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

## 1. Notes

### 1.1. General instructions

Storage temperature: +5 - 30 °C

### 1.2. Product identifiers

2-propanol [67-63-0], Didecyldimethylammonium chloride [7173-51-5], Benzyl-C12-16-alkyldimethylammonium chloride [68424-85-1], C12-14-Alkyl [(ethylphenyl)methyl]dimethyl ammonium chloride [85409-23-0]

### 1.3. Explanation of the symbols

**1** - CE mark with notified body; **2** - Medical device; **3** - Item number; **4** - Batch designation; **5** - Name and address of the authorized representative's registered office in Switzerland; **6** - Unique identifier of a medical device; **7** - UDI marking with standard com-

pliant HIBC data content; **8** - Expiry date; **9** - Manufacturer; **10** - Follow instructions for use; **11** - Temperature limit; **12** - Quantity information; **13** - Responsible for the translation; **14** - Observe instructions for use; **15** - Use hand protection; **16** - Protect from heat / Protect from sunlight; **17** - Protect from moisture / Store in a dry place; **18** - Package orientation at the top; **19** - Stacking limit; **20** - Information.

### 1.4. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik.

## 2. Intended purpose

Solution for disinfection of surfaces of non-invasive medical devices.

**Indication:** Surface disinfection of non-invasive medical devices.

**Contraindication:** Not for using for Stainless steel, which is prone to chloride corrosion.

**Target group:** Professionally trained health workers.

## 3. Safety-related information

### 3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

### 3.2. Danger and safety information

The mixture is classified as non-hazardous according to (EC) regulation no. 1272/2008 (CLP).

### 3.3. Risks and side effects

This product does not have direct skin contact with the user or the patient.

### 3.4. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

#### General instructions:

Immediately remove contaminated, saturated clothing.

#### After inhalation:

In case of irritation of the respiratory tract, consult a doctor.

#### After skin contact:

In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.

### After eye contact:

If the product gets into the eyes, rinse immediately with plenty of water for at least 5 minutes with the eyelid held wide open. Then consult an ophthalmologist.

### After ingestion:

Rinse mouth immediately and drink plenty of water.

### 3.5. Personal protective equipment (PPE)

**Hand protection:** Suitable gloves type:

Suitable material: NBR (nitrile rubber). Breakthrough time: >480 min. Thickness of the glove material: 0.4 mm. Additional hand protection measures: Wear cotton undergloves if possible.

## 4. Product description

Alcohol-free spray and wipe disinfection.

### 4.1. Composition

Spray and wipe disinfection based on quaternary ammonium compounds.

## 5. Use

### Application of spray disinfection:

Spray the undiluted solution directly onto the surface to be disinfected and keep it moist for the duration of the exposure time.

### Application wipe disinfection:

Put the undiluted solution on a wipe. Then completely wet the surface to be disinfected and keep it moist for the duration of the exposure time.

## 6. Disposal

Dispose of product in compliance with official regulations.

### - before intended use:

Product waste code: 070699; Hazardous waste: No; Waste designation: Waste not otherwise specified

### - after intended use:

Product waste code Packaging 070699; Hazardous waste: No; Waste designation: Waste not otherwise specified

## 7. Annex

### 7.1. Order numbers and scope of delivery

**122000050** - GREEN&CLEAN SK, 6 x 750 ml bottles + 2 x foam nozzle

**122000051** - GREEN&CLEAN SK, 1 x 5 l canister

**122000064** - GREEN&CLEAN SK, 1 x 750 ml bottle with foam nozzle

### 7.2. Change history

Revision 200005329, 06.07.2023: Newly created.

Revision 200005329v01, 29.08.2024: Pictogram „Observe instructions for use“ replaced.

Corrected various spelling and translation errors. Subject to printing and setting errors!

Spectrum of action and exposure time	Standard	Test information	Exposure time
bactericidal	EN 13727	Low load	1 min
levurocidal	EN 13624	Low load	1 min
4-field test (bactericidal, levurocidal)	EN 16615	Low load	5 min
limited virucidal	EN 14476   EN 16777	Low load	1 min

### Traduction de la notice originale

Tetras GmbH, Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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**METASYS Medizintechnik GmbH**