GREEN&CLEAN H1

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

1.1. General instructions

UN 1760

Storage temperature: +5 °C - +30 °C

Product identifiers:

Subtilisin (9014-01-1)

Quaternary ammonium compounds, benzyl-C8-18-alkyldimethyl, chlorides (63449-41-2)

Alkyl polyglycol ether C10-16 with PO and EO (69227-22-1)

1.2. Explanation of the symbols

| | |
|----|--|
| 1 | CE mark |
| 2 | Medical device |
| 3 | Unique identifier of a medical device |
| 4 | UDI marking with standard compliant HIBC data content |
| 5 | Item number |
| 6 | Batch designation |
| 7 | Expiry date |
| 8 | Manufacturer |
| 9 | Follow instructions for use |
| 10 | Name and address of the authorized representative's registered office in Switzerland |
| 11 | Temperature limit |
| 12 | Quantity information |
| 13 | Responsible for the translation |
| 14 | Health hazard |
| 15 | Corrosive |
| 16 | Dangerous for the environment |
| 17 | Harmful to health |
| 18 | Observe instructions for use |
| 19 | Use hand protection |
| 20 | Use eye protection |
| 21 | Wear protective clothing |
| 22 | Protect from heat / Protect from sunlight |
| 23 | Protect from moisture / Store in a dry place |
| 24 | Fragile |
| 25 | Package orientation at the top |
| 26 | Limited quantity |
| 27 | Stacking limit |
| 28 | Information |

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik.

2. Intended purpose

GREEN&CLEAN H1 is a decontamination concentrate for the automatic cleaning and decontamination of the METASYS H1 hygiene system.

Indication: Decontamination and cleaning of dental suction systems during dental treatment.

Contraindication: Not applicable.

Target group: Dentally trained health personnel.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

ANGER

| | |
|--------------------|--|
| H302 | Harmful if swallowed. |
| H312 | Harmful in contact with skin. |
| H314 | Causes severe skin burns and eye damage. |
| H334 | May cause allergy or asthma symptoms or breathing difficulties if inhaled. |
| H400 | Very toxic to aquatic life. |
| P101 | If medical advice is needed, have product container or label at hand. |
| P102 | Keep out of reach of children. |
| P260 | Do not breathe dust/fume/gas/mist/vapours/spray. |
| P261 | Avoid breathing dust/fume/gas/mist/vapours/spray. |
| P273 | Avoid release to the environment. |
| P280 | Wear protective gloves/protective clothing/eye protection/face protection. |
| P301+ P330 + P331 | IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. |
| P303 + P361 + P353 | IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water (or shower). |
| P305 + P351 + P338 | IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. |
| P310 | Immediately call a POISON CENTER/doctor/... |
| P304 + P340 | IF INHALED: Remove person to fresh air and keep comfortable for breathing. |
| P342 + P311 | If experiencing respiratory symptoms: Call a POISON CENTER/doctor. |
| P391 | Collect spillage. |
| P501 | Dispose of contents/container to a suitable recycling or disposal facility. |

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

General instructions

Immediately remove contaminated, saturated clothing.

After inhalation

In the event of breathing difficulties, give oxygen. In the event of irregular breathing or respiratory arrest, initiate artificial respiration. Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label.

After skin contact

In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.

After eye contact

In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist.

After ingestion

Do not induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

Indication of any immediate medical attention and special treatment needed

Information for the doctor: Symptomatic treatment.

3.4. Personal protective equipment (PPE)

Protection and hygiene measures

Avoid contact with skin, eyes and clothing. Soiled garments must be washed before reuse. Immediately remove contaminated, saturated clothing.

Eye/Face protection

Suitable eye protection: Tightly sealing safety goggles

Recommended eye protection brands: DIN EN 165

Hand protection

Suitable material: NBR (nitrile rubber).

Required properties: liquid-tight.

Thickness of the glove material: 0.4 mm

Recommended glove brands: EN ISO 374

Additional hand protection measures: Wear cotton undergloves if possible.

Note: Breakthrough times and swelling properties of the material must be taken into account.

Physical protection

Suitable physical protection: Laboratory coat

Respiratory protection

Respiratory protection is not required when used properly and under normal conditions.

3.5. Combination with other products / materials

The GREEN&CLEAN H1 product is used together with the METASYS H1 hygiene system.

4. Product description

4.1. Product description

GREEN&CLEAN H1 is a concentrate based on quaternary ammonium compounds, special surfactants and enzymes for decontamination of the METASYS H1 hygiene system. It is automatically dosed by the H1 hygiene system during the aspiration process.

4.2. Composition

Quaternary ammonium compounds, special surfactants, enzymes

4.3. Exposure time

The application time is determined by the cleaning cycles of the H1 hygiene

system.

5. Use

5.1. Dosage

In the H1 hygiene system, dosing is automated.

5.2. Application

For application in the H1 hygiene system, the instructions for use of the dosing device must be observed.

6. Disposal

Dispose of product in compliance with official regulations.

| Product | before intended use |
|--------------------|-------------------------------|
| Product waste code | 70699 |
| Hazardous waste | No |
| Waste designation | Waste not otherwise specified |

| Packaging | after intended use |
|--------------------|--------------------|
| Product waste code | 150102 |
| Hazardous waste | No |
| Waste designation | Plastic packaging |

7. Annex

7.1. Order numbers and scope of delivery

| Order number | Designation |
|--------------|-----------------|
| 122000005 | 4 x 500 ml bag |
| 122000006 | 30 x 500 ml bag |

7.2. Change history

| Revision | Date | Description |
|-----------|------------|----------------|
| 200004288 | 11.04.2024 | Newly created. |

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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METASYS Medizintechnik GmbH