GREEN&CLEAN MB

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EN – Instructions for use

1. Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

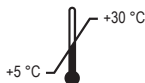
1. Notes

1.1. General instructions

DENTAL USE

Storage temperature:

Product identifiers:

Laurylpyridinium chloride (104-74-5)

Alcohol, C10-16, ethoxylated, propoxylated (69227-22-1)

1.2. Explanation of the symbols

1	CE mark	13	Use eye protection
2	Medical device	14	Information
3	Item number	15	Protect from heat / Protect from sunlight
4	Batch designation	16	Protect from moisture / Store in a dry place
5	Expiry date	17	Package orientation at the top
6	Manufacturer	18	Stacking limit
7	Observe instructions for use	19	Health hazard
8	Temperature limit	20	Unique identifier of a medical device
9	Quantity information	21	UDI marking with standard compliant HIBC data content
10	Name and address of the authorized representative's registered office in Switzerland		
11	Responsible for the translation		
12	Use hand protection		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended purpose

GREEN&CLEAN MB is a citric acid and enzyme-based solution for cleaning and masking unwanted odours of the spittoon of dental treatment units.

Indication: Removal of blood and saliva in the spittoon.

Contraindication: Not applicable.

Target group: Professionally trained health workers.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

H319	Causes serious eye irritation.
H411	Toxic to aquatic life with long lasting effects.
P264	Wash thoroughly after handling.
P270	Do not eat, drink or smoke when using this product.

WARNING

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures:

General instructions: Immediately remove contaminated, saturated clothing.

After inhalation: In the event of irritation of the respiratory tract, consult a doctor.

After skin contact: Then rinse with soap and water.

After eye contact: If product gets into the eye, keep eyelid open and rinse immediately with large quantities of water, for at least 5 minutes. Subsequently consult an ophthalmologist.

After ingestion: Rinse mouth immediately and drink plenty of water.

3.4. Personal protective equipment (PPE)

Eye/Face protection: Suitable eye protection: Tightly sealing safety goggles. Recommended eye protection brands: DIN EN 165

Hand protection: Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm. Recommen-

ded glove brands: EN ISO 374. Additional hand protection measures: Wear cotton undergloves if possible.

Respiratory protection: Respiratory protection is not required when used properly and under normal conditions.

4. Product description

4.1. Product description

Citric acid and enzyme-based spittoon cleaner for cleaning and masking unwanted odours of the spittoon of dental treatment units.

4.2. Composition

Citric acid, quaternary ammonium compounds, non-ionic surfactants and enzymes

4.3. Exposure time

The application time depends on the degree of soiling of the spittoon, but should be at least 30 seconds.

5. Use

5.1. Dosage

The product GREEN&CLEAN MB is a user solution that does not need to be diluted.

5.2. Application

Note:

- > Use GREEN&CLEAN MB undiluted!
- > Wear personal protective equipment during application!

-
- | | |
|----|--|
| 22 | Fill the cleaning solution into the dispenser bottle. Use a funnel (PE or PP) for easier handling. |
| 23 | Screw the brush head with dosing device onto the dispenser bottle. |
-

Application:

-
- | | |
|----|---|
| 24 | Turn dispenser bottle upside down for dosing (press 1 x). |
| 25 | Clean the rinsing basin with the brush. |
| 26 | Then rinse the spittoon thoroughly to remove contamination. |
-

6. Care and maintenance

6.1. Regular cleaning measures

Note:

- > Regularly disinfect the brush head in an ultrasonic or instrument bath (do not autoclave!)
- > Replace brush heads regularly

Clean the brush head

27	Clean and disinfect the outer surfaces of the application bottle with a surface disinfectant.
28	Remove the brush head. Replace the brush head if it is heavily soiled or worn.
29	Pre-clean the brush head with water and then disinfect it in an ultrasonic or instrument bath.

7. Recycling and disposal

Dispose of product in compliance with official regulations.

	before intended use	after intended use
	Product	Packaging
Product waste code	70699	150102
Hazardous waste	No	No
Waste designation	Waste not otherwise specified	Plastic packaging

8. Annex**8.1. Order numbers and scope of delivery**

Order number	Designation
122000035	GREEN&CLEAN MB Start, 1 x 180 ml dispensing bottle with brush
122000032	GREEN&CLEAN MB Intro kit, 2 x 500 ml + 2 dispensing bottles+ 2 brushes
122000033	GREEN&CLEAN MB Refill kit 1, 4 x 500 ml + 4 brushes
122000034	GREEN&CLEAN MB Refill kit 2, 4 x 500 ml
122000037	GREEN&CLEAN MB replacement brushes, 4 pieces
121000020	GREEN&CLEAN MB replacement dispensing bottle, Dispensing bottle with brush, empty

8.2. Change history

Revision	Date	Description
200005320	22.06.2021	Newly created.
200005320v01	16.09.2024	Correction Chapter 1.3. to English; typing error in Chapter 4.7; correction of REF and order numbers. New order numbers.

Subject to printing and setting errors!

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1. Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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